


Mouth Gag for cattle

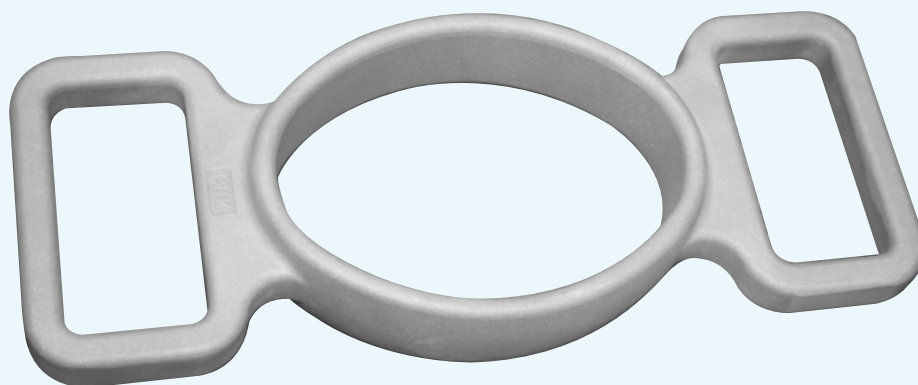


【Feature】

- Keep the cow opening the mouth for intraoral treatment or catheter insertion, medicine application, etc.
- Light weight & easy to carry
- Frost finish to avoid reflection for taking photos

【How to use】

1. Wash and disinfect the mouth gag.
2. Make sure to restrain the cow.
3. Open the mouth to insert the gag toward the molar and keep opening
4. Do necessary treatment
5. After treatment, slowly remove the gag.
6. Wash the gag before mucus is dried.

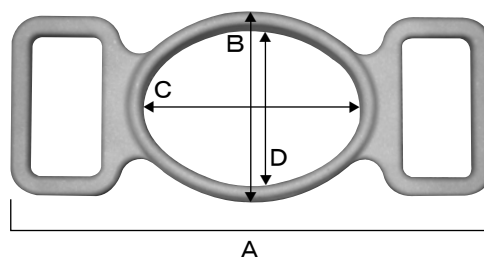

Code No.	05003200
Size	A 240mm/B96mm/C110mm/D80mm
Weight	380g
Material	Aluminium

16-12-500S